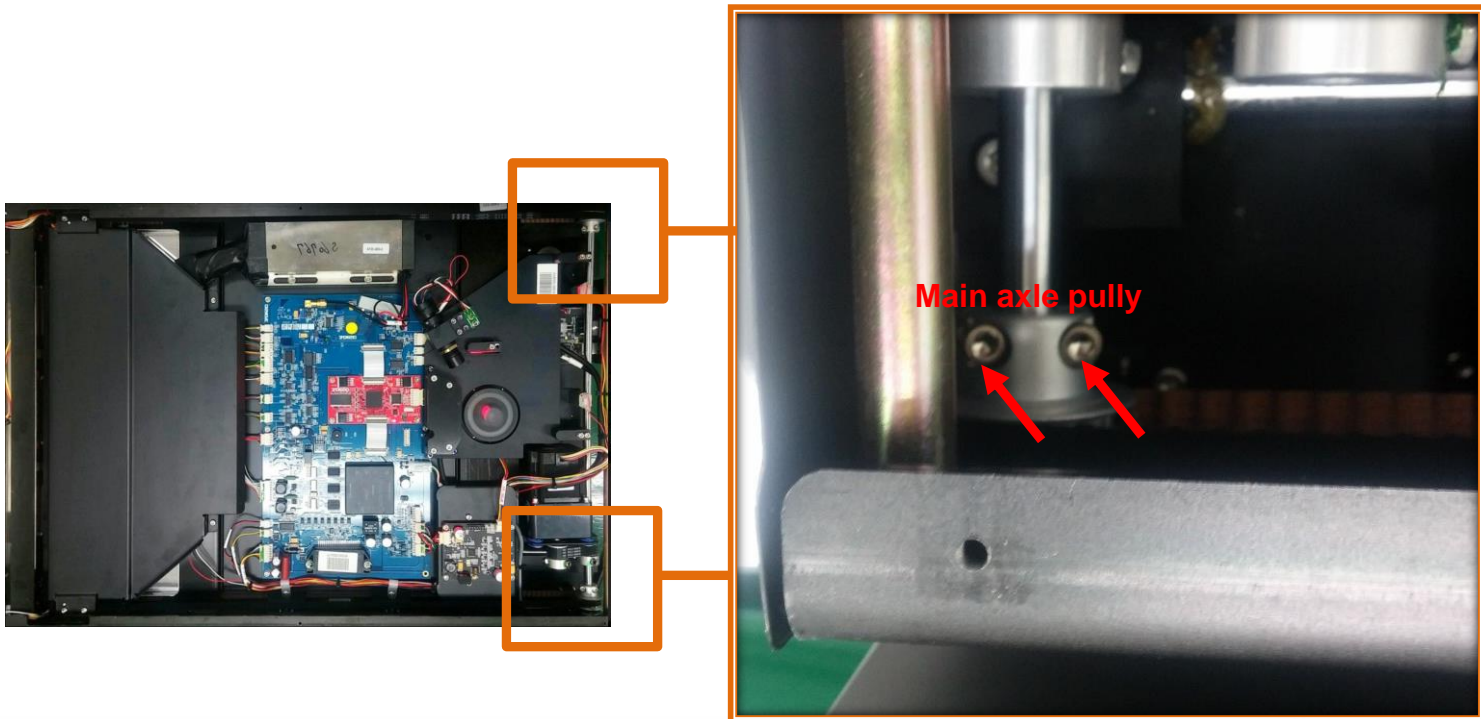

How to Align the Stage

How to align the stage

1. Remove Top Cover. (Refer to the service manual)
2. Remove Internal Cover. (Refer to the service manual)
3. Run Quantor+ as administrator rights.
4. Open up the Scanner control window (pw:fireadmin).
5. Click "Forward" and then stop when you access the two screws on the each side of main axle pulley.



How to align the stage

7. Switch off the power

6. Loosen the two screws on each main axle pulley.

Do not unscrew completely.

7. Switch on the power.

The stage automatically moves to the home position.

8. Tighten two screws on the left main axle pulley when the stage is at the home position.

9. Tighten two screws on the right main axle pulley while a stage is pushed toward the front section of the scanner. (Please refer to the video on the way of tightening.)

