



❖ How to export database files from old computer

- At first, you have to export the DB file from the old computer.

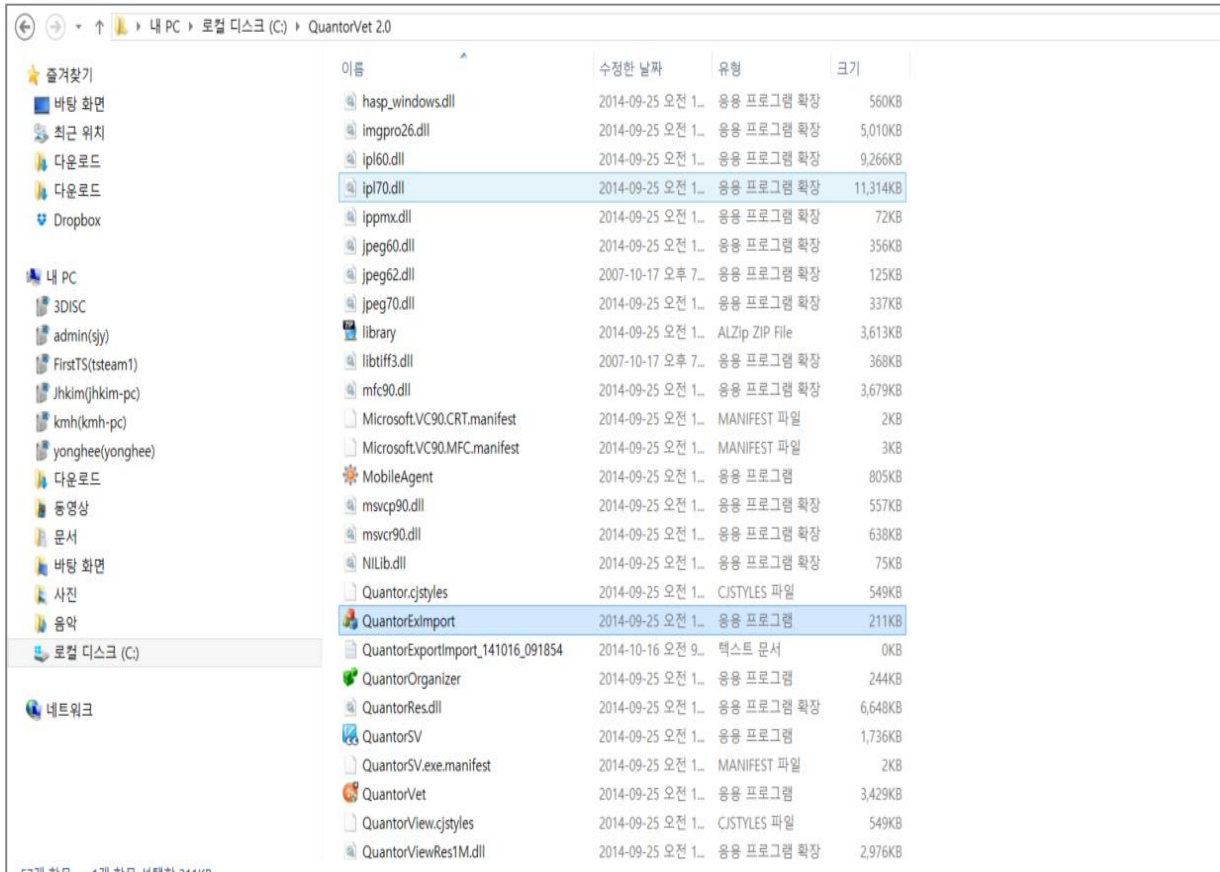


1. Go to the QuantorVet folder
2. Double click the QuantorExlImport program.
3. Pop-up the new window

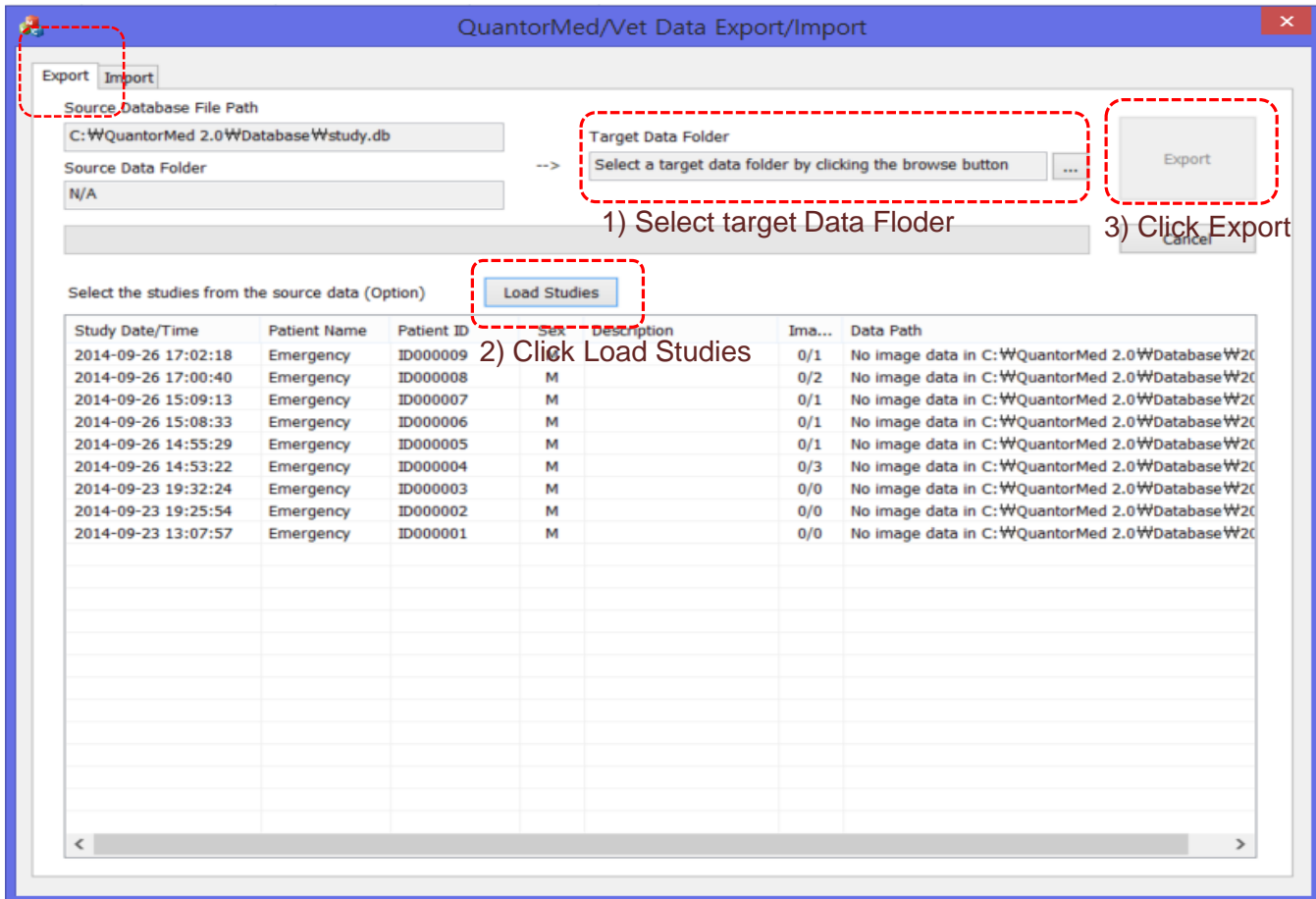

1. Select "Export" tab
2. Select the "Target Data Folder"
 - Database file will be stored in the target folder that you select.
1. Click "Load Studies"
 - Show up all the list
3. Click "Export"

❖ How to import database files to new computer

- you have to import the DB file to the new computer..

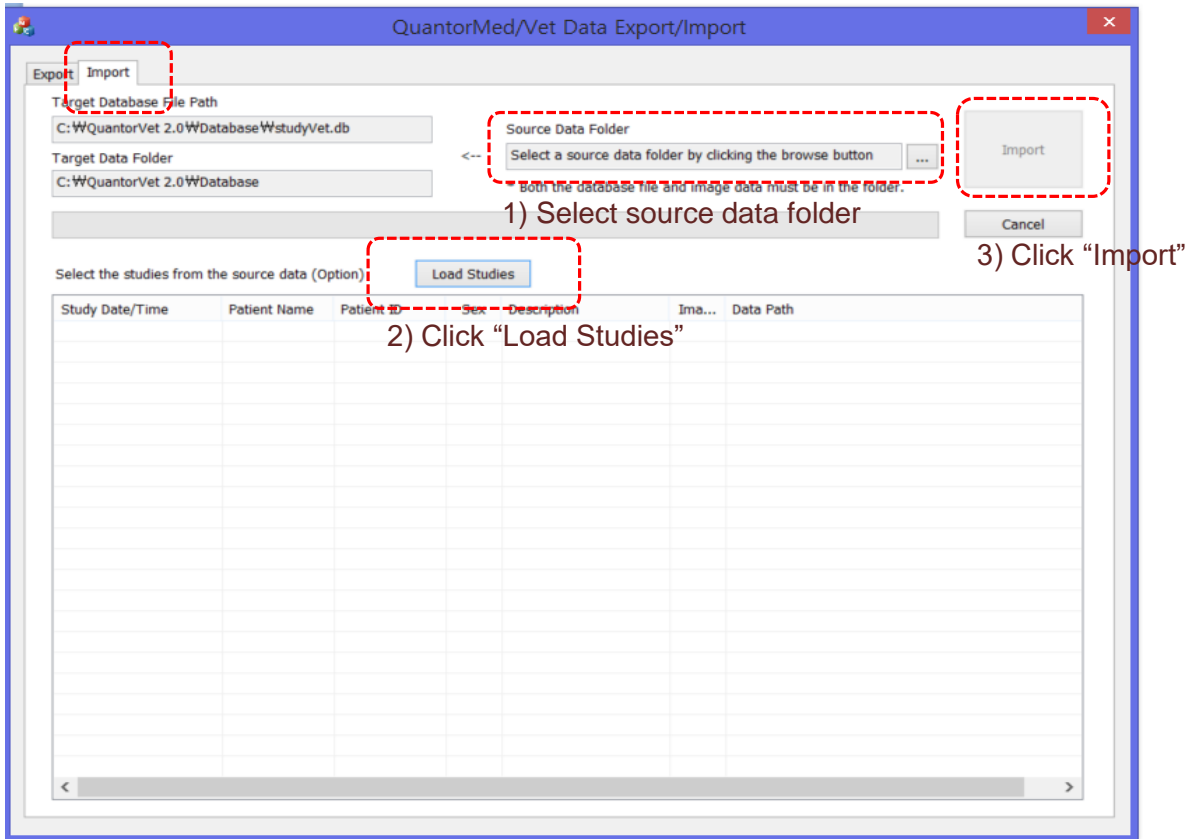

1. Select "Import" tab
2. Select the "Source Data Folder"
 - Database file will be imported to target folder from source data folder.
3. Click "Load Studies"
4. Click "Import"